



# ALTA DIAGNOSTICS, INC.

2560 BUSINESS PARKWAY STE C, MINDEN, NV 89423  
 (800) 359-9691 (775) 267-3001 FAX: (775) 267-1142

**18 MONTH  
OPEN VIAL  
STABILITY**

## LIQUID URINE CONTROL

### FOR MICROALBUMIN

Level 1:105921A  
Expires March 2012

Level 2: 105922A  
Expires March 2012

	QUANTITATIVE ASSAY		MICRAL CHEMSTRIP*		PROCEDURE
	Level 1	Level 2	Level 1	Level 2	
<b>UNITS</b>	<b>Mg/L</b>	<b>Mg/L</b>	<b>Mg/L</b>	<b>Mg/L</b>	1. For dipstick use, remove cap and insert dipstick.  2. Remove dipstick after the time recommended by the dipstick manufacturer.  3. Follow the procedure for testing as described by the manufacturer of the dipstick.  4. If the vial will be used within 30 days you may recap the control and leave it at ambient temperature (15-25 degrees C). IF The bottle is used beyond 30 days, store the bottle at 2-8 degrees C.  5. Any changes in the reagents, methods or instrument methodology by the manufacturer may result in different values.  Consult manufacturer's instructions for the procedure for further information.  * Chemstrip is a registered product of Roche.
<b>RANGE</b>	<b>6 ± 5.5</b>	<b>40 ± 20</b>	<b>0 - 20</b>	<b>20 - 100</b>	
DAY 1					
DAY 2					
DAY 3					
DAY 4					
DAY 5					
DAY 6					
DAY 7					
DAY 8					
DAY 9					
DAY 10					
DAY 11					
DAY 12					
DAY 13					
DAY 14					
DAY 15					
DAY 16					
DAY 17					
DAY 18					
DAY 19					
DAY 20					
DAY 21					
DAY 22					
DAY 23					
DAY 24					
DAY 25					
DAY 26					
DAY 27					
DAY 28					
DAY 29					
DAY 30					
DAY 31					
<b>TOTAL</b>					
<b>MEAN</b>					